



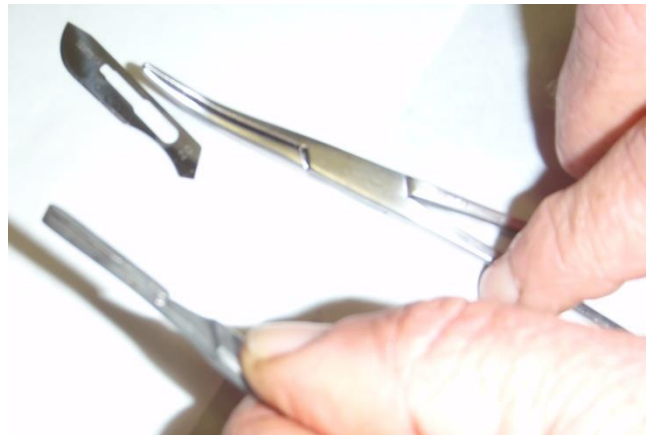

# Preparing Spay Packs

## A. Clean instruments:

1. ***Put all instruments in open position.*** This is very important as autoclave heat will damage instruments in closed position.
2. Remove scalpel blades – place blades in cold sterile solution tray.

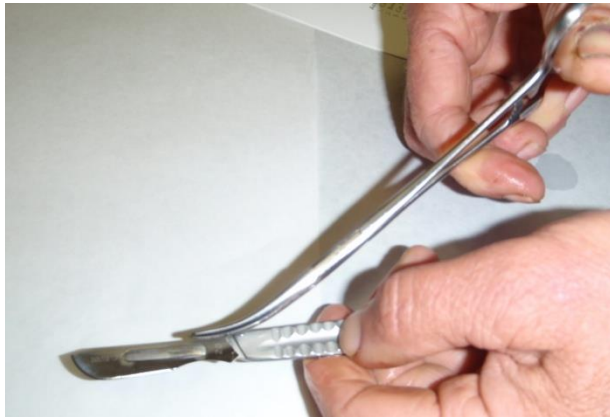


3. Place fine suture material in dry tray.
4. Wash instruments in soapy water. Use toothbrush to clean, especially serrations and hinge areas. Wash out blade slot.
5. Rinse well with tap water while opening and closing hinges.



# Preparing Spay Packs

## B. Assemble pack:

Layout items for spay pack on a clean towel:

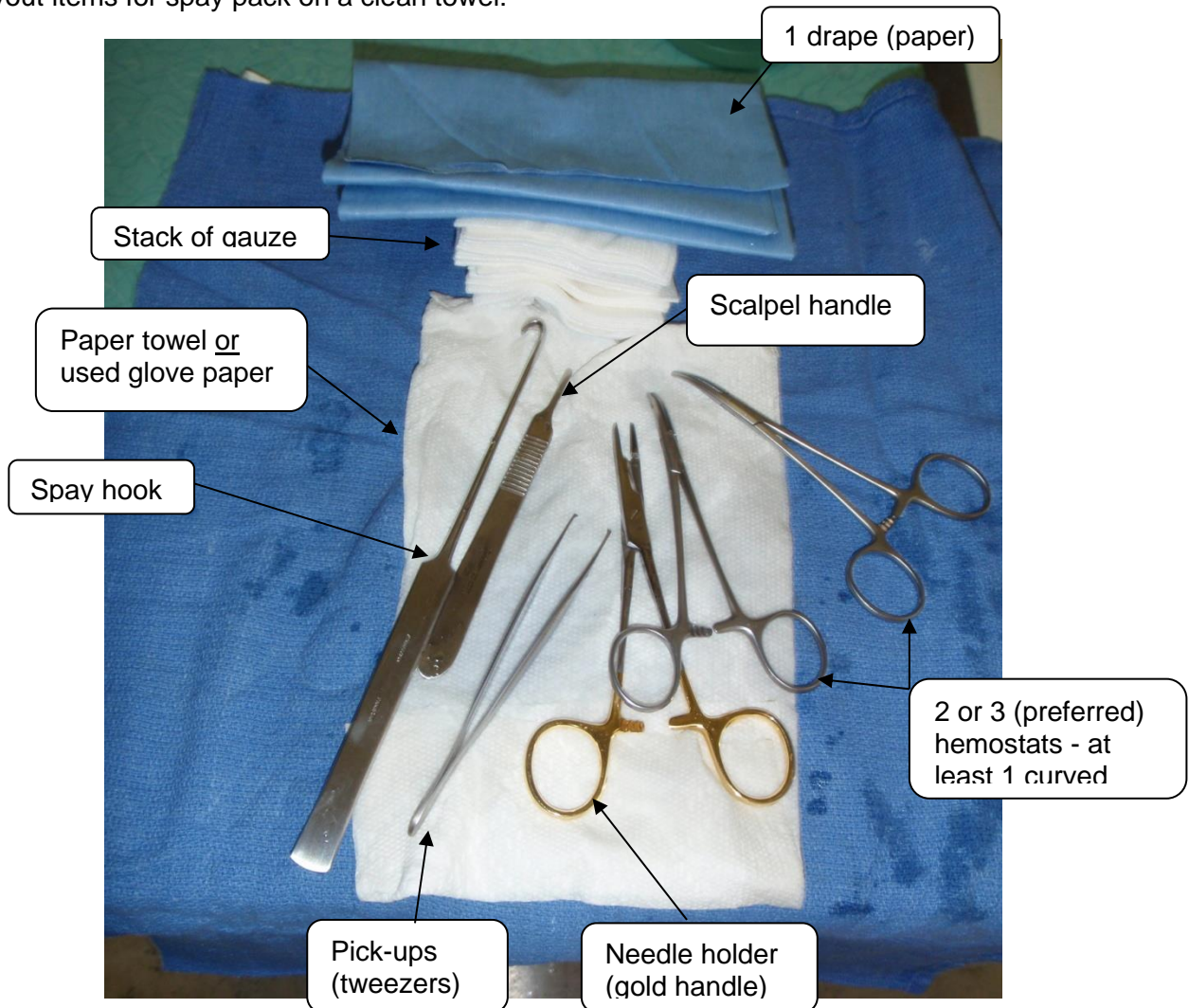




# Preparing Spay Packs

## C. Fold pack closed:

Fold pack closed so that when opened in surgery, no item inside needs to be touched:

Fold one edge  
over contents

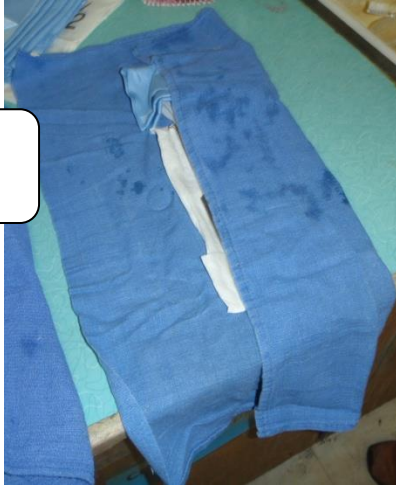

Fold other  
edge over  
first



Fold ends to close



Place a piece of  
autoclave tape to  
hold close

